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 Thanks to its completely computerized function checking system, AUTOCCLAVE Mod. ASTER 2000 class B having fractional vacuum is able to sterilize all the following materials:

Class B:
 Massive and porous products, double wrapped, unwrapped, wrapped, "A" and "B" types in conformity with EN 13060:2004 rules.

GENERAL FEATURES:

- Fractional vacuum autoclave type B that makes possible the sterilization of the following materials: hollow body A/B, porous, massive, wrapped and unwrapped.
- Microprocessor check.
- Multilanguage display.
- Program selection menu. (See "Sterilization cycles table")
- Alphanumerical Display (16 x 2).
- Digital on-screen display for selected cycle,
- Program selection with 5 keys: Start/Enter, Cycle, Dry Off, Pump, Reset
- LED for warning: LEV.MIN, LEV.USED, LEV.MAX, ALARM, DOOR LOCKED, CYCLE.
- Last cycle read in and start with Start key.
- Last cycle execution repetition in case of blackout.
- Real-time on screen display of the following values:
 - Chamber temperature and pressure.
 - Current phase.
 - Steam generator temperature and chamber shell temperature.
 - Chamber shell temperature set and pressure set.

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Through the sequential pressure of the key "PUMP".

- Stand by: immediate or absent.
- Double tank: clean and dirty water.
- Water to be used: demineralised/distilled.
- Water consumption: 850 cc per cycle
- Possible cycles with max. load: n. 6 cycles
- Possibility of setting automatic and manual load/discharge of the tanks.
- Hard vacuum with two-head pump
- Stainless steel chamber.
- Chamber dimensions: 20 LT, Ø 267 mm, Depth: 390 mm.

Safety devices:

- Checking system always on.
- Process Evaluation System.
- Electromechanical safety system against accidental opening of the door during the sterilization cycles.
- Steam generator with safety thermostat.
- Pre-heating chamber resistance having thermostat with manual reset.
- 2,4 bar calibrated safety valve (certified).
- Steam generator separated from the chamber.
- Certified bacteriological filter.
- Integrated printer with thermo-graphic paper. Maximum paper consumption per cycle: 300 mm. PC serial exit RS 232. Cycle's data recording (every 0,5 sec.) with Hyper Terminal (text document). You will also have the opportunity to create calculation charts (for example: Excel, Open office, Star office and so on).
- Electrical feeding: 230 V (AC) / 50-60 Hz
- Maximum absorbed power: 1950 W.
- Feeding current: 8,8 A
- Overall dimensions: W 555 L 720 H 520 mm

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STERILIZATION CYCLES:

PROGRAM	STERILIZATION TIME	DRYING TIME	PRESSURE	TOTAL TIME	NOTES
VACUUM TEST	-	-	-0,85 bar	20 minutes	Empty chamber
HELIX TEST	-	-	-	50 minutes	Spiral system according to EN 867-5:2004 with chemical indicator Browne Helix test according to EN 867-5:2001
BOWIE & DICK TEST	-	-	-	50 minutes	Bowie & Dick Pack according to EN 867
121 °C S30 A13 Hollow, wrapped, porous.	30 minutes	13 minutes	1,1 bar	90 minutes	See table 1 Medium charge 3Kg
121 °C S20 A13 Hollow, solid, wrapped and non wrapped.	20 minutes	13 minutes	1,1 bar	75 minutes	See table 1 Medium charge 3Kg
134 °C S18 A13 Hollow, wrapped, porous. PRIONE	18 minutes	13 minutes	2,1 bar	75 minutes	See table 1 Medium charge 3Kg
134 °C S04 A13 Hollow, wrapped, solid	4 minutes	13 minutes	2,1 bar	65 minutes	See table 1 Medium charge 3Kg
134 °C S03 A03 Solid. Emergency cycle "Flash"	3 minutes	3 minutes	2,1 bar	25 minutes	Maximum charge. 1 Kg.

Chamber Pre-heating Exclusion at high temperature

The chamber pre-heating starts immediately at an high temperature by setting the "Absent" Stand-By, at the switch on. Otherwise it is possible to delay the switch off of the pre-heating by setting a "Delay" for a minimum of 0 to a maximum of 90 minutes.

While, by setting the "Immediate" Stand-By, the chamber is not pre-heated at an elevated temperature and the optimal chamber temperature is reached after 10 Minutes from cycle start at the ambient temperature of 20°C.

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