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Following to the coming into effect of the EUROPEAN SET OF RULES UNI:EN:13060-2004 "Small steam sterilizer units", TITANOX S.r.l. is very glad to present you the new AUTOCLAVE Mod. ASTER 3000 class B-S-N having fractional vacuum.

Thanks to its completely computerized function checking system, AUTOCLAVE Mod. ASTER 3000 is able to sterilize all the following materials:

- Class B:**
Massive and porous products, wrapped and unwrapped.
- Class S:**
A and B hollow body products, wrapped and unwrapped.
- Class N:**
Massive unwrapped products.

GENERAL FEATURES:

- Fractional vacuum autoclave type B – S – N that makes possible the sterilization of the following materials: hollow body A/B, porous, massive, wrapped and unwrapped.
- Microprocessor check.
- Multilanguage display.
- Software realized to help the operator during the cycle selection. The software will help the operator to choose the correct cycle for the material to be sterilized. Program selection menu. (See "Sterilization cycles table")
- 128-character display (8 x 16).
- Digital on-screen display for: selected cycle, phases, temperature, pressure, cycle counter, device serial number and dater.

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- Program selection with 4 keys: Start/Enter, Reset, Up/+, Down/-.
- Last cycle read in and start with Start key.
- Programmable start-up.
- Stand by: immediate, absent or after a period of time with energy saving.
- Double tank: clean and dirty water.
- Water to be utilized: demineralised.
- Water not recycled for the sterilization cycle.
- Water consumption: 500 cc per cycle (type N) / 900 cc per cycle (type B and S).
- Possible cycles with max. load: n. 5 cycles type B/S; n. 9 cycles type N.
- Possibility of setting automatic and manual load/discharge of the reservoirs.
- Hard vacuum with two-head pump.
- Stainless steel chamber.
- Chamber dimensions: 30 LT, 267 mm, Depth: 544 mm.
- Security devices:
 - Checking system always on.
 - Process Evaluation System.
 - Electromechanical safety system against accidental opening of the door during the sterilization cycles.
 - Steam generator with security thermostat and external resistance with manual reset.
 - 2,4 bar calibrated security valve (certified).
 - Electronic chamber pressure excess control . When the pressure will reach 2,5 bar, the control card will immediately stop the current cycle and an alarm will be heard.
- Steam generator separated from the chamber.
- Certified bacteriological filter.
- Integrated printer with thermo-graphic paper. Maximum paper consumption per cycle: 300 mm.
- PC serial exit RS 232. Cycle's data recording (every 0,5 sec.) with Hyper Terminal (text document). You will also have the opportunity to create Excel charts.
- Electrical feeding: 230 V (AC) / 50-60 Hz
- Maximum absorbed power: 1950 W.
- Feeding current: 8,8 A
- Overall dimensions: W 555 L 720 H 520 mm.

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TOTAL CYCLE TIME WITHOUT PRE-HEATING PHASE

You have the opportunity to start the cycle without waiting pre-heating phase.

The optimal chamber temperature is reached after 10 Minutes from cycle start at the ambient temperature of 20°C.

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